



Vac Lid Urine Collection Device

Sterile • 3oz. • 400 Cups per Case

item # TNR HXK263VL

(Collection tube not included)

- Lower shipping costs - more tubes can be shipped than containers
- Do away with leakage problems during shipping
- Simplified process of cup to tube transfer
- Urine specimen test tube goes directly into instrument when applicable
- SAVE SPACE with tubes on racks rather than specimen cups
- Dispose in sharps container to lower urine specimen waste disposal cost




STEP 1

After collection, put lid on cup.



STEP 2

Peel off protective label. Warning: Sharp Needle!

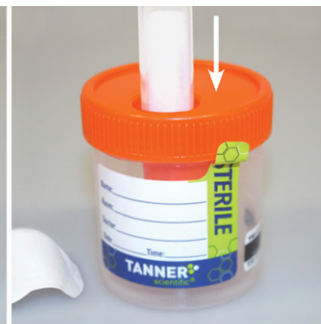

STEP 3

Push vacuum tube into cavity until cap has been fully penetrated and hold in position.



STEP 4

Remove tube after collection. Reattach label on lid.

Mercedes
MEDICAL

PHONE: 800.331.2716 | FAX: 800.359.8807 | WEB: MERCEDESMEDICAL.COM